## Parcus Cannulated Drills and Headed Reamers

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---

## **Parcus Cannulated Drills and Headed Reamers**

### **1. Indications:**

Parcus Cannulated Drills and Headed Reamers are used to create tunnels and sockets in bone. This product is designed to be used in conjunction with a 2.4mm guide pin.

### **2. Warnings:**

- A. This product is intended for use by or on the order of a physician.
- B. Never place this product over a bent or damaged guide pin.
- C. Placing off-axis loads on the guide pin by deflecting the drill/reamer may cause damage to both the drill/reamer and the guide pin.
- D. Never use a dull or damaged drill/reamer.
- E. This product is not intended to be left in the body.
- F. Be sure to remove any debris from the cannulation of the drill/reamer during the cleaning process.

### **3. Material:**

These products are manufactured from a stainless steel alloy. The material used in the manufacture of the devices that are intended to

be placed inside the body are radio-opaque and can, therefore, be detected with conventional X-Ray or fluoroscopy.

**4. Sterilization and Cleaning:**

This device is not provided sterile. It should be cleaned and sterilized per facility standards using validated protocols.

**5. Directions for use:**

- A. Use a purpose designed drill motor or wire driver to place a guide pin in the desired position.
- B. Securely attach the drill/reamer into a drill chuck.
- C. Place the drill/reamer over the guide pin.
- D. Make sure that the drill motor is set to rotate the drill/reamer in a clockwise direction.
- E. Prudently advance the drill/reamer over the guide pin, ensuring to maintain a coaxial orientation between the pin and the drill/reamer.
- F. After the desired depth has been reached, reverse the drill motor and remove the drill/reamer from the bone.

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