Parcus Drill Guide

Important Product Information

Produktinformationsblatt

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Parcus Drill Guide

1. Indications:

The Parcus Drill Guide is used to assist in the placement of guide pins in bone.

2. Warnings:

- A. This product is intended for use by or on the order of a physician.
- B. Never use a bent or damaged guide pin with this device.
- C. No component of this instrument is intended to be left in the body.
- D. Be sure to remove any debris from the cannulation of the drill sleeve during the cleaning process.

3. Material:

This product is manufactured from a stainless steel alloy and PEEK. The materials used in the manufacture of this device that are intended to be placed inside the body are radio-opaque and can, therefore, be detected with conventional X-Ray or fluoroscopy.

4. Sterilization and Cleaning:

This device is not provided sterile. It should be cleaned and sterilized per facility standards using validated protocols.

5. Directions for use:

- A. Assemble the Parcus Drill Guide by placing the Adjustable Tibial Guide Slide on the Tibial Guide Arc, insert the selected Tibial Guide Arm into the guide slide and insert the pin guide sleeve into to the ratcheting pin boss on the Tibial Guide Arc.
- B. Set the Tibial Guide Slide to place the pin at the desired angle relative to the top of the Adjustable Tibial Guide Slide.
- C. Index the guide arm such that the laser mark on the Tibial Guide Arm corresponds to the laser mark on the Tibial Guide Slide.
- D. Advance the pin guide sleeve by pushing the sleeve into the ratcheting pin boss on the Tibial Guide Arc. The flat on guide sleeve engages the ratcheting action when placed in the 12:00 position. Rotating the guide sleeve 90 degrees (3:00 or 9:00) or squeezing the trigger mechanism on the inside of the guide pin boss, will release the ratcheting lock.

- E. Select a guide pin 2.4mm in diameter and place it in a purpose designed drill. Make sure that the pin is of adequate length to pass through the Guide Pin Sleeve and advance to the tip of the Tibial Guide Arm.
- F. Advance the guide pin to the tip of the Tibial Guide Arm.
- G. Release the ratcheting mechanism for the Guide Pin Sleeve, as described above, and remove the guide, leaving the guide pin in the desired position.

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