Parcus 5.5 – 6.5mm Punch/Tap with Handle

Important Product Information

Produktinformationsblatt
Información Importante sobre el Producto
Informations Importantes sur le Produit
Informações Importantes sobre o Produto

Directions for Use

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Parcus 5.5 – 6.5mm Punch/Tap with Handle

1. Indications:

The Parcus 5.5 – 6.5mm Punch/Tap with Handle is designed to prepare bone for the implantation of the Parcus 5.5mm and 6.5mm V-LoX™ PEEK CF Suture Anchors.

2. Warnings:

- A. This product is intended for use by or on the order of a physician.
- B. This product should not be used to implant any device other than the Parcus V-LoX PEEK CF Suture Anchors.
- C. Do not use this product if it is bent or there is visible damage to the tip or thread form.

3. Material:

This device is manufactured from a Stainless Steel Alloy shaft and an Aluminum handle. The materials used in the manufacture of this device that are intended to be placed inside the body are radio-opaque and can, therefore, be detected with conventional X-Ray or fluoroscopy.

4. Cleaning:

- A. Immediate rinsing and cleaning after use with an enzymatic detergent will effectively remove and prevent drying of adherent blood, tissue, etc.

- B. Scrub device with a soft brush, paying special attention to areas where debris might accumulate. Always avoid any harsh materials that can scratch or mar the surface of the device.
- C. Rinse the device thoroughly with water following the cleaning process.

5. Sterilization:

This device is not provided sterile. It should be cleaned and sterilized per facility standards using validated protocols.

6. Directions for use:

- A. Identify the site for anchor insertion and gently tap the Parcus Punch/Tap into the bone until the threads of the Punch/Tap can be engaged.
- B. Rotate the Punch/Tap clockwise into the bone until the top of the laser line is flush with the bone surface.
- C. Carefully remove the Punch/Tap by rotating the handle in a counter-clockwise direction until its threaded portion is clear of the bone.
- D. Insert a Parcus V-LoX PEEK CF Suture Anchor into the prepared hole.

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