## Parcus Non-cannulated Headed Reamers

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## **Parcus Non-cannulated Headed Reamers (English)**

### **1. Indications**

Parcus Non-cannulated Headed Reamers are designed to be used to create sockets and tunnels in bone.

### **2. Warnings**

- A. This product is intended for use by or on the order of a physician.
- B. This product is supplied sterile. Do not use this product if the packaging has become wet or damaged in any way.
- C. This product is designed for Single Use Only and should not be re-sterilized.
- D. Never use a bent or damaged reamer.
- E. This product is not intended to be left in the body.

### **3. Material Specifications**

This product is made from a stainless steel alloy. The material used in the manufacture of this device is radio-opaque, and can be detected with conventional X-ray or Fluoroscopy.

### **4. Packaging and Labeling**

- A. Check the sterile packaging for damage of any kind.
- B. Do not use the product if the sterile packaging has been compromised in any way.

### **5. Sterilization**

This product is supplied in sterile packaging. The contents are sterilized by EO gas. These products must never be re-sterilized.

### **6. Storage**

Products must be stored in the original unopened package in a dry place and must not be used beyond the expiration date indicated on the package.

### **7. Directions for Use**

- A. Select and verify a Non-cannulated Reamer of appropriate diameter.
- B. Open the product using aseptic technique.
- C. Place the non-fluted end of the reamer in a purpose designed drill motor or reamer.
- D. Place the tip of the Non-Cannulated Reamer at the site for the creation of the tunnel or socket.
- E. Prudently advance the Non-cannulated Reamer into the bone to the desired depth, using the drill motor or reamer set to rotate in a clockwise direction.
- F. Withdraw the Non-cannulated Reamer from the bone, remove it from the chuck of the drill motor or reamer and dispose it in an appropriate manner.

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